

Protecspine Assembly Instruction

PARTS:



Base



Power Block



Motor



Cushion (2)



Spacer Rings



3 Hole Plate




Main Pipe



**Long Bar for
Shoulder Rope**




Rope assembly and Strap

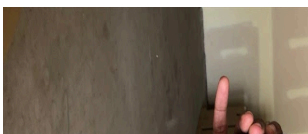



**Cervical locking device
with digital scale**

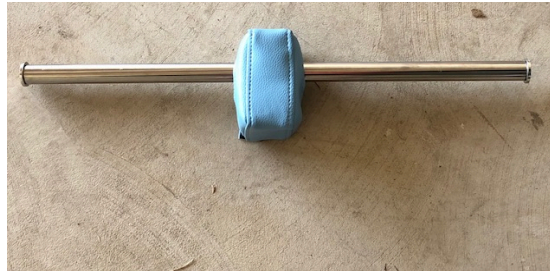

Short Bar Assembly for leg Rope



Head-rest



Set a control box on base frame with the four screw.



**Set the motor on the mounting holes on Protec
base frame, tighten the bolts with the allen wrench.**




**Make sure to inserting the remote control
wire connector to a middle of connector hole.**



**Insert the waist arm and slide it .
down into Protec poles
Insert spacer rings
between two waist cusions. (The buckles shall be on right side for both cusions)**



**After setting up 2 waist arms into poles .
then insert the 3 hole flat plate**

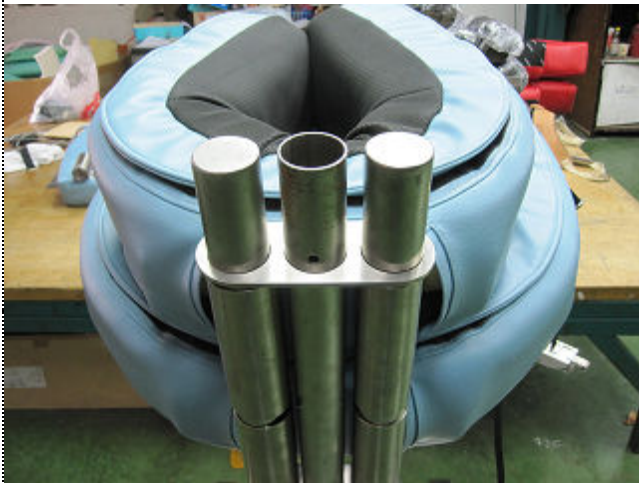

**Push into 3 hole flat plate all the way
on the waist cushion.**



For the Main Pipe:



**First remove the set-screw from the main pipe.
Attach the pipe stand into the middle
of pole and screw in the set screw as well as tighten the 4 bolts with the Allen Wrench while making sure that the pole is centered.**



Set the head rest unit into 2 poles.



**There are three parts that need to be
installed on the main pipe.
First slide thru the shoulder mobilization**




base and install the long rope bar on it. Install the rope assembly. (Notice the orientation of the bar)



Second slide thru the cervical locking device with digital scale. The lever shall be in up position to move the locking assembly.



Slide into a cervical block to pipe stand when lever in up position.

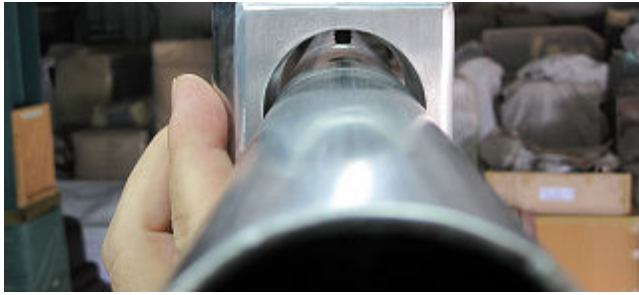

**Down position of the cervical block lever to lock the position.
The cervical block has a digital scale assembly installed.**



Lastly, open up the short bar assembly and install in the front mounting.

Set the mobilizer unit on pipe stand.

.....



Put a top cover over the center block. (Head Protection)



Set the mobilizer pulley assembly on each side of pipe.



Hang the Legs and Arms leather straps on to the pulley units.

The longer straps are for the legs and the shorter one for the arms.